

## FISIOMEDIC

- Pains articulate, arthritides
- Bursitis, Dislocations, Sprains
- Muscular pains
- Convulsions, Miosites, Tendinites
- Tendinites
- Sport traumas
- Muscular fatigue
- Cervical pains
- Lumbar pains
- Illnesses of the back
- Distrofia Reflexa
- Mental fatigue



### CARES OF USE

Don't operate your mat, for more than 2 hours continually.

Don't use the massage mat in wet places.

When it uses the key of FUNÇÃO, don't use the same program for more than 3 minutes in the maximum force or more than 10 minutes in the force medium or minimum.

When it is necessary to substitute the fuse, only for other with the same specifications.

When it finishes a session, leave the mat, turned off, at least 30 minutes.

After finishing the use of the mat, disconnect the source of the energy.

For cleaning, remove the connection of energy, before.

Don't use strong or corrosive products of cleaning. Use a cloth lightly humid.

Conserve the mat in dry and fresh place.

The use of the massage mat is not recommended in women pregnant, physical deficient, people with heart problems or with processes ulcerous, postoperative. In spite of the therapeutic applications, the massage mat, Fisiomedic, is not a product of medical treatment.

**In case of doubts, ask to your doctor.**

The massage mat Fisiomedic, was developed and manufactured with top technology and safety. Follow the use recommendations and conservation of this mat strictly.


### TECHNICAL INDICATIONS

1. Connect the source **1** to the energy. The source is bi-voltage.

(It comes selected of factory for 220V. To alter in case the voltage is of 110V.)

2. Display of the remote control.

- LIGA DESLIGA** ON/OFF switch
- PONTOS** Selection of the program time for a massage session.
- TEMPO** Time massage program.
- FUNÇÃO** Selection of the punctual location of the massage.
- +** It reduces or it increases the intensity of the vibration.

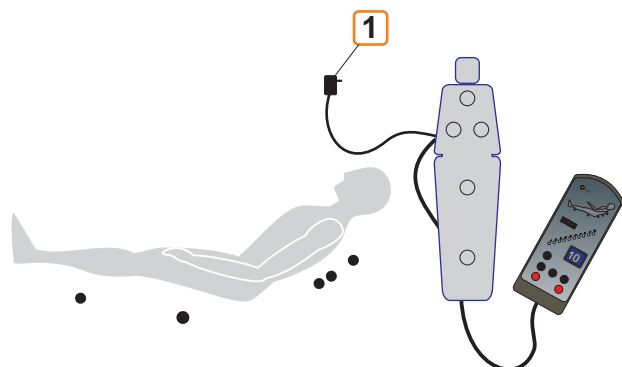

**MAX DENTAL, LDA.**

504734326

comercial@max-dental.com

max-dental@max-dental.com

maxdental-lx@max-dental.com

● HEAD-OFFICE - Urb. Bom João - Chasfa, Lt. 4 - R/C - E - Traz. 8005-321 FARO - PORTUGAL (00351) TEL 289863848 - FAX 289863045 - TLM 966469550

● CENTRO - Av. Almirante Reis, 20-20E - 1º B 1150-017 - LISBOA - PORTUGAL (00351) TEL 218141898 - FAX 218142188 - TLM 969568425 - 969567645

● SPAIN - (0034) Mobile 661712464 - ehingerm@hotmail.com

